

MASTDECIDE On Farm Clinical Mastitis Culture Test

**RRP £9.15
per sample**

**Less than the cost
of 3 mastitis tubes**

What do you need?

- **Cleanly taken Clinical milk sample**
- **Mastdecide Test Kit.**
- **Small egg incubator (cost £35—£40)**

What is entailed?

- **5 Minutes preparation**
- **8 - 14 Hours incubation**
- **Rapid reliable results to aid Antibiotic treatment decision making**

Simple interpretation

- **3 diagnostic options**
and

- **2 treatment options**
based on media colour

Double pink - No growth

NSAID only - no antibiotic

Single pink - Gram negative

NSAID only - no antibiotic

Double white - Gram positive

Antibiotic and NSAID

ValeLab are importing and distributing Mastdecide within the UK.

Mandy Boddy Laboratory manager



**Ring or email for an
information pack,
professional price and
carriage charges**

August 2020

Mastdecide Rapid On Farm Culture (OFC)

Rapid

Less risk of reduced cure rates in antibiotic treated cases

Incubation time 8 to 14 hours compared to other OFC systems which are often 24 hours - treatment delay of one milking rather than 2 milkings

Easy to read and Cost effective


Only if both test vials turn white, is there a need for antibiotic treatment. Potentially saving of antibiotic use in up to 50 % of clinical mastitis cases and helping farmers achieve responsible use of antibiotics.

Each mastitis case should be treated with a zero milk withhold NSAID so mastitis cases not receiving antibiotic will not need a prolonged medicine withdrawal period. Less antibiotic treatment, less antimicrobial resistance risk and the potential to return the milk to the tank quicker.

MastDecide is therefore an exciting new and progressive mastitis treatment decision aid. It is cost effective, rapid and reliable and therefore is a must have for any modern dairy vet and farmer.

Please contact Andrew Biggs BVSc FRCVS for further details and to find out the benefits of Mastdecide and how we and other practices are using it successfully.


“Double Pink”

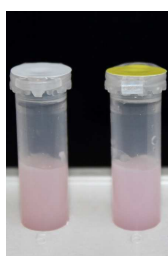

“Single Pink”



“Double White”

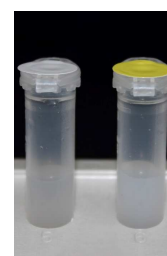

The Vale Veterinary Laboratory

**The Laurels,
Station Road,
Tiverton, Devon,**

EX16 4LF

Phone: 01884 259584

Email: ValeLab@btinternet.com

**Targeted antibiotic use (tLCT)
Less antibiotic treatments
No need for long milk withholds
Milk returned to tank earlier
Less milk discarded which is
good for the environment
More milk sold - more profit**

One box comes with 10 tests and equipment (pipettes, vials, wipes etc.) We can spilt boxes and sell test separately